COMPOSITION

Arginine, phosphate de calcium, fluorure de sodium

PROPRIETES

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1.010.002-03_005
Rev. 2021-10

COMPOSITION

Arginine, calcium phosphate, sodium fluoride

PROPERTIES

DESENSETIN suspension is used for treatment of hard tooth tissues for reduction of sensitivity. This water-based material contains arginine and calcium phosphate. After applying on the tooth surface and rubbing the suspension into defect, arginine, having a high buffering capacity, under the influence of saliva, forms a complex compound, which blocks acid attacks and reduces the pain. Calcium phosphate, getting into defect, blocks access of irritants and serves as a source of ions which are necessary for remineralization of dental hard tissues.

INDICATIONS

- Dental hard tissues hyperesthesia reduction
- Prevention of caries

CONTRAINDICATIONS

Drug idiosyncrasy
Use with great care for allergic patient
Do not misuse

SIDE EFFECTS

No known side effects as far as the application of product is concerned

METHOD OF USE

Attention! It is recommended to store material at room temperature (22–24) °C within 1–2 hours before use.

Before use of **SUSPENSION**, clean, disinfect and dry injured dentine. **SUSPENSION** should be applied with brush (or any other applicator) on problem areas of the tooth and intensively rubbed into injured surface. Additional amount of suspension could be applied additionally without any risk of previous portion destruction.

Patient is recommended to restrain from food, from rinsing the mouth within 2 hours after procedure, to avoid acid drinks and fruits within 3–5 days after procedure.

To get a good result use **DESENSETIN** suspension 2–3 times a year.

To consolidate the effect, or in case of unexpected occurrence of sensitivity, procedure can be carried out by patient itself at home. For home use of **DESENSETIN** suspension, it is recommended for patient to rub suspension with finger in sensitive areas of the tooth after cleaning. It is also recommended to rinse mouth with water, without much effort, and do not eat for 2 hours.

PRESENTATION

Suspension (syringe / vial)	5 ml
Suspension (vial)	15 ml

STORAGE

Store at the temperature from 5 °C to 25 °C
Do not use after expiry date
Shelf life — 3 years

PRECAUTIONS

Wear suitable protective clothing, gloves and goggles before use.
Avoid contact with eyes. In case of eye or skin contact, as well as contact with soft tissues and mucosa of oral cavity, remove the material immediately and wash with water.

Use with caution in patients with mental disorders.

Do not use material for pregnant women (because of insufficient clinical data).

In cases of allergic reactions in particularly sensitive patients, the material should be removed and its further application abandoned.

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TehnoDent Co. Ltd,